


The position of both the **arm shell and elbow support** is **vital** to ensuring a good operation of the ORTHOPUS device

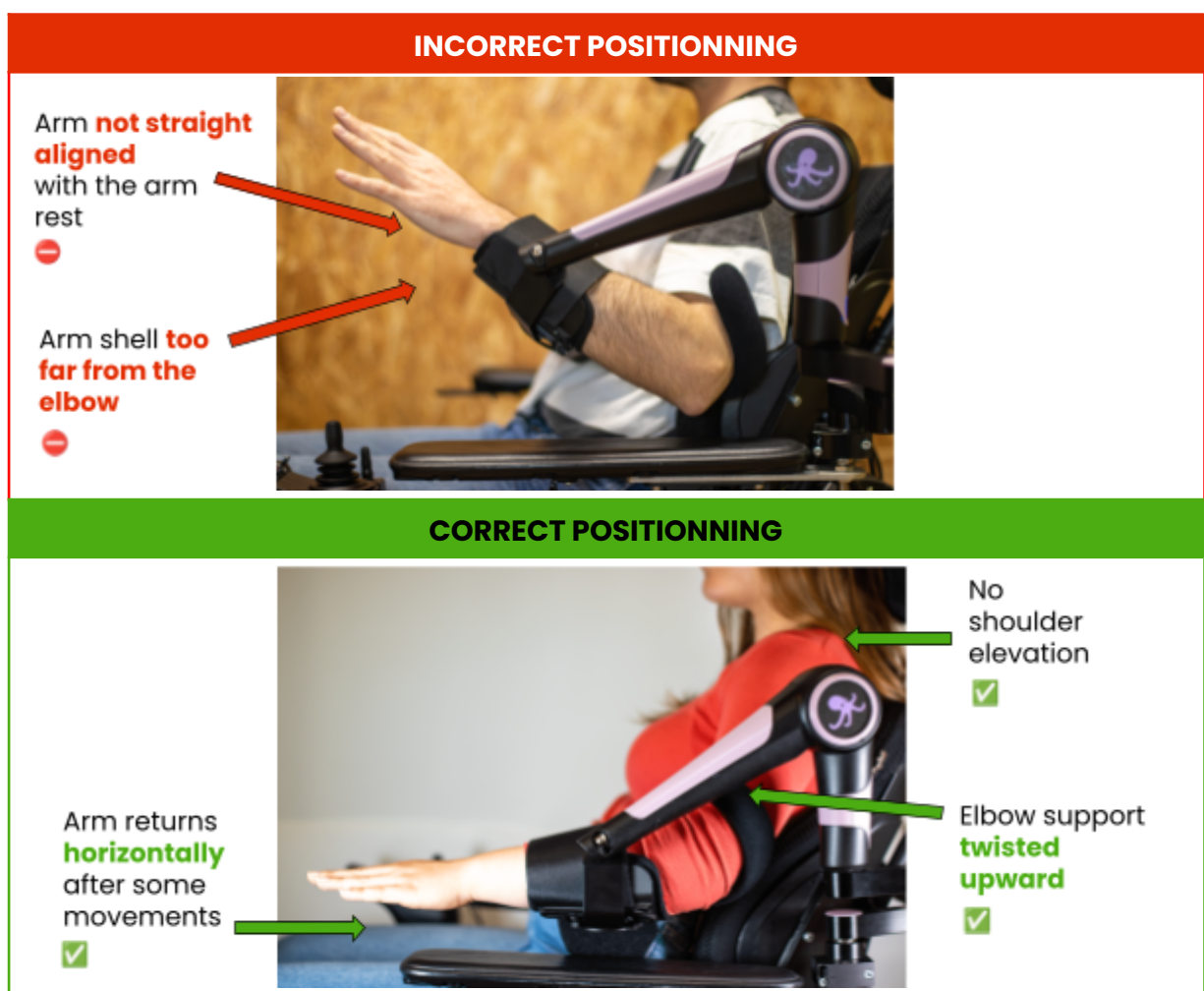

FIX THE ARM SHELL & ELBOW SUPPORT POSITIONNING



2 outer screws

→ to set up arm shell position



2 inside screws

→ to set up elbow support position

ADJUST THE ELBOW SUPPORT POSITIONNING



Twist the elbow support to push it **upwards**. The flat part must be **in full contact** with the arm.



Feel free to remove the user's arm from the arm brace to do this more easily.



Find **all the detailed information on ORTHOPUS' arm brace (4 sizes, adjustments)** in the Instructions For Use in the ORTHOPUS packaging box or on orthopus.com/documentation



Scan this QR code to watch our video **TUTORIAL - Installation and adjustment of the arm brace** (7min)